


Universal Applicator


To load the tag, depress the metal clip and insert the female tag under the clamp.

1

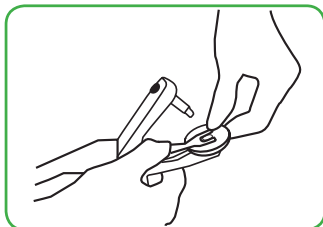


Place the male tag completely onto the applicator pin. It is recommended to dip the applicator holding the ear tags into antiseptic or disinfectant solution to avoid the risk of infection.

2

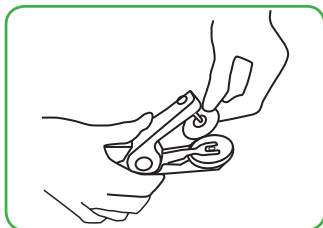

Position the tag between the two main cartilages, approximately halfway between the animal's head and the tip of the ear. Then squeeze handles firmly to join the male and female tag together.


3

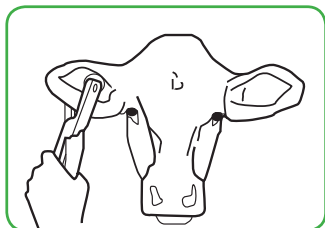

Important:

Always apply the male tag entering from the back (outside) of the ear. Ear tags must be positioned at the center of the ear and between the main two rows of ear cartilage

!

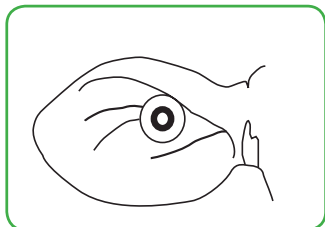

Datamars Australia

1300 594 696

Ph. +61 7 3635 8222

www.datamars.com.au/agri-id