



## Slim NLIS tag Application Instructions

- Slim NLIS tags are applied with a Universal applicator.
- If there is a black insert in the applicator<sup>1</sup>, please remove it before applying the NLIS tags.

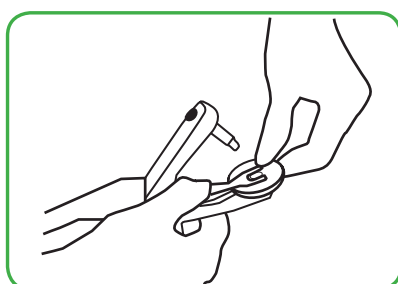

# 1


To load the tag, depress the metal clip and insert the female tag under the clamp.

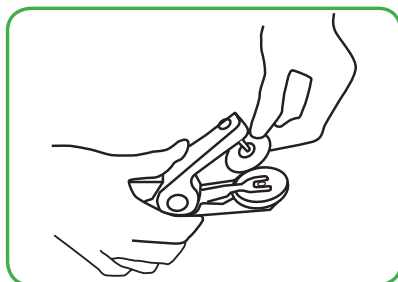

# 2


Place the male tag completely onto the applicator pin.

It is recommended to dip the applicator holding the ear tags into antiseptic or disinfectant solution to avoid the risk of infection.

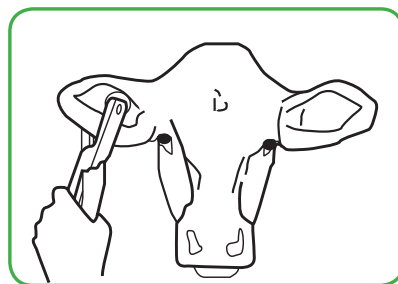

# 3

Position the tag between the two main cartilages, approximately halfway between the animal's head and the tip of the ear. Then squeeze handles firmly to join the male and female tag together.

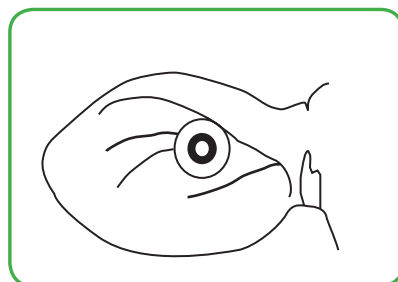

# !

### Important:

Always apply the male tag entering from the back (outside) of the ear. Ear tags must be positioned at the centre of the ear and between the main two rows of ear cartilage.

<sup>1</sup>The Universal applicator requires an insert to apply two piece management tags from certain brands, however, the insert is not required to apply Agri-ID NLIS tags. The insert is positioned on the bottom jaw of the tagger.